AMARYLL MANUAL BREASTPUMP

Instructions for use | Instrucciones de uso

 ARDO

1. Intended use

The Ardo Amaryll breastpump is intended to be used by lactating women to express and collect milk from their breast. The unit is intended for single users only.

2. Possible problems and troubleshooting

Weak or no vacuum:

- Ensure that the Amaryll manual breastpump has been correctly assembled and is not damaged.
- Check whether the lip valve is cracked or stuck together.
- Check that the casing cover seals well.

Pain when pumping:

- Ensure that the nipple is in the middle of the breast shell.
- Check that the breast shell size fits your nipples.
- If problems continue seek professional help.

3. User risks

There are no known contraindications.

Safety regulations: Only use after cleaning (see section on "Cleaning"). After cleaning, check for any damage.

4. Technical Information


M	Materials: Polypropylene (PP) / Silicone
	Disposal: can be safely disposed of in compliance with local regulations (Non-hazardous waste)
	Packaging: Caution, fragile

	Keep packaging away from heat
	Keep packaging away from moisture
Preferably store the Optiflow in a dark place (silicone)	

Instructions for use in other languages are available at www.ardomedical.com

6.2 Pumping

- Wash your hands, sit down comfortably and relax.
- Turn the casing cover to the desired position.
- Centre the nipple so that, when pumping, it moves freely in the breast shell / breast shell insert, and pump by activating the lever.
- Adjusting the suction intensity: the further you press the lever, the greater the suction intensity.
- Adjusting the speed interval: the faster you move the lever up and down, the higher the pumping frequency.
- Our recommendation: begin at a high speed interval and a low suction intensity in order to stimulate the milk ejection reflex. Once the milk is flowing, reduce the speed interval and increase the suction intensity.

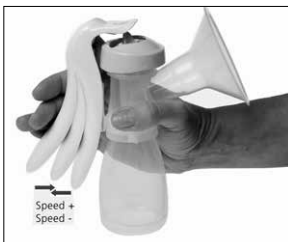

6.3 Choice of Breast Shell and Optiflow Massage insert

If your nipple rubs uncomfortably on the inner side of the funnel, use a larger breast shell / larger breast shell insert. If the areola is also sucked into the funnel, use a smaller breast shell. The Optiflow Massage insert can only be used with the 31 mm breast shell.

28 mm breast shell insert

Optiflow

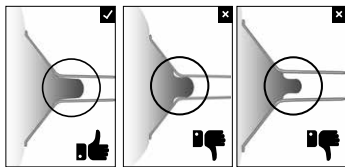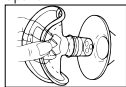7. Storage of breast milk

Check with your hospital for specific instructions!

At room temperature: 6 hours at max. 77 °F (25 °C); in the refrigerator: 5 days at max. 39 °F (4 °C); in the freezer compartment of the refrigerator (3 °F / -16 °C) 3 - 6 months; in the deep-freezer (< 0 °F / < -18 °C) 6 - 12 months.

8. Cleaning

Before first use, disassemble and boil all components for 10 minutes. Air dry before assembling and use.

After every use, wash with soapy water and rinse well. Once a day, disassemble and boil the components for 10 minutes, or put the components in the dishwasher, then dry and cover with a clean cloth. Do not clean the lip valve with a brush. The lever, casing cover and bottle stand can, but need not, be boiled. After cleaning, check for any damage.

CAUTION: Let components cool after boiling before touching them.

5. Product description

Amaryll manual breastpump:

- a.) Lever
- b.) Casing Cover
- c.+d.) Membrane pot with membrane con rod
- e.) 26 mm breast shell
- f.) 1 lip valve plus 1 spare valve
- g.) 150 ml bottle
- h.) Bottle stand
- i.) Bottle cap
- j.) Cap liner
- k.) Cleaning brush
- l.) 31 mm breast shell
- m.) 28 mm breast shell insert
- n.) 26 mm Optiflow massage insert

Amaryll Essentials Manual Breastpump:

from a to j (excluding spare lip valve)

Amaryll Kombikit:

from a to d (excluding spare lip valve)


WARNING: CHOKING HAZARD – Small parts included. Keep small parts away from babies and children.

6. Instructions for use

6.1 Assembly

Important: boil before first use (see section on cleaning).

- 1.) Assemble according to the instructions.
- 2.) Important: place the membrane pot securely over the groove of the membrane con rod. (Membrane pot: the smooth side of the membrane pot should face outwards).
- 3.) Put on the casing cover, so that one mark of the casing cover lies on top of the mark of the breast-shell, then rotate the casing cover about 0.5 cm clockwise
 - 3.1) standard Position
 - 3.2) lever position left side
 - 3.3) lever position right side
- 4.) The lever is mounted on the membrane con rod and by means of light pressure downwards clicks into the casing cover.

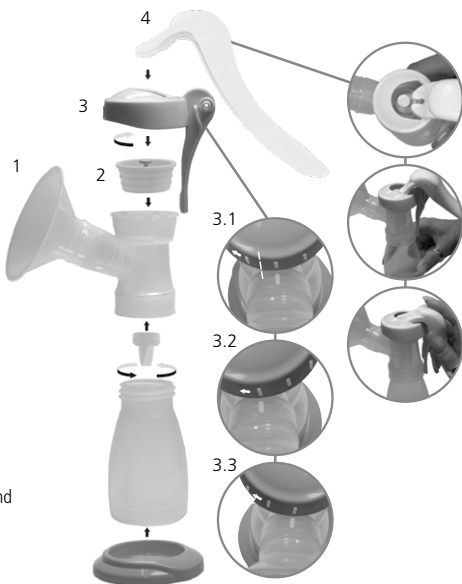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Ardo medical AG